

# Evaluation of the Sebia Hydrigel 5 von Willebrand factor (VWF) assay with an in-house agarose gel electrophoresis method for VWF multimer analysis

Goodfellow KJ<sup>1</sup>, Bowyer AE<sup>1</sup>, Nouadje G<sup>2</sup>, Beaulieu G<sup>2</sup>, Kitchen S<sup>1</sup>, Makris M<sup>1</sup>  
<sup>1</sup>Sheffield Haemophilia and Thrombosis Centre, UK <sup>2</sup> Sebia, France

## INTRODUCTION

VWF multimer analysis is essential to characterise the subtype of VWF abnormalities, along with assays such as VWF activity/antigen, factor VIII, factor VIII binding and genetic testing.

Multimer analysis is currently laborious, non-standardised and interpretation of gel results subjective.

## OBJECTIVES

The aim of this study was to determine whether the Sebia Hydrigel semi-automated system will improve our interpretation of results and reduce hands-on staff time, as our in-house method takes 4 days to produce results and are interpreted visually.

Normal and patient samples were analysed in parallel with both the in-house method and the Sebia Hydrigel 5 VWF multimer assay. VWF:Ag and VWF:Act assays were carried out using Siemens reagents.



## METHODS

### In-house multimer method :

- 1.6 % SDS Agarose gel electrophoresis
- Visualisation using alkaline phosphatase -conjugated antibody
- Low resolution gel used for identification of types
- High resolution gels used to further categorise type 2A
- Total hands on time: 190 mins over 4 days


### Sebia Hydrigel 5 VWF multimers with Hydrasys 2 Scan instrument :

- Electrophoresis in 2.0% SDS agarose gel
- Direct immunofixation and visualisation with peroxidase-labelled antibody
- Followed by densitometry
- Total hands-on and assay steps = 6 hours and 40 mins

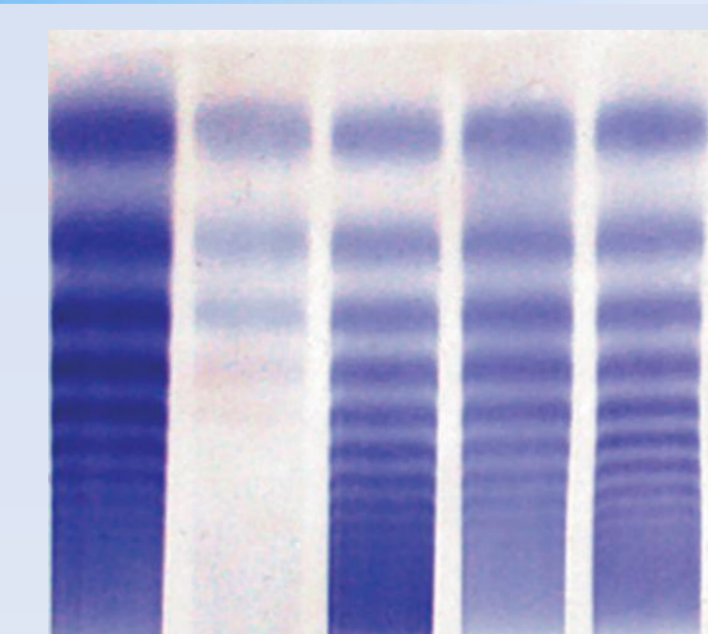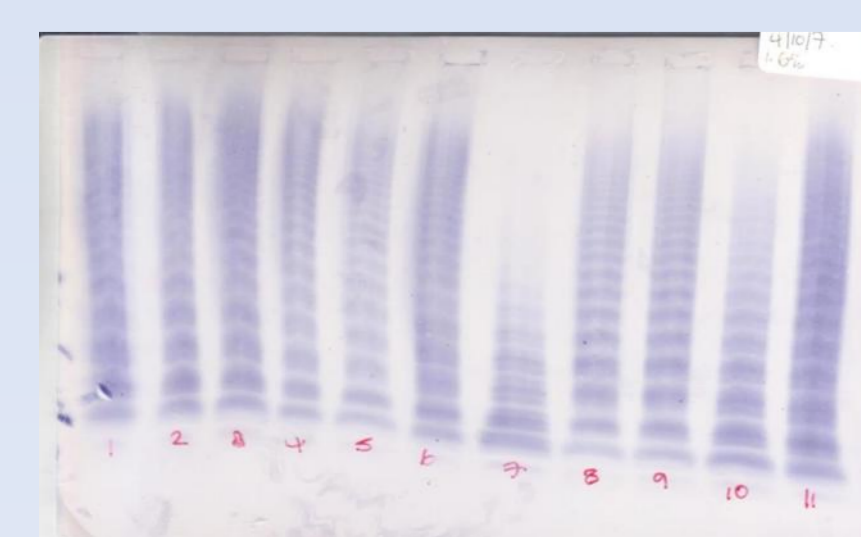


## RESULTS


### Normal Donors : n=23

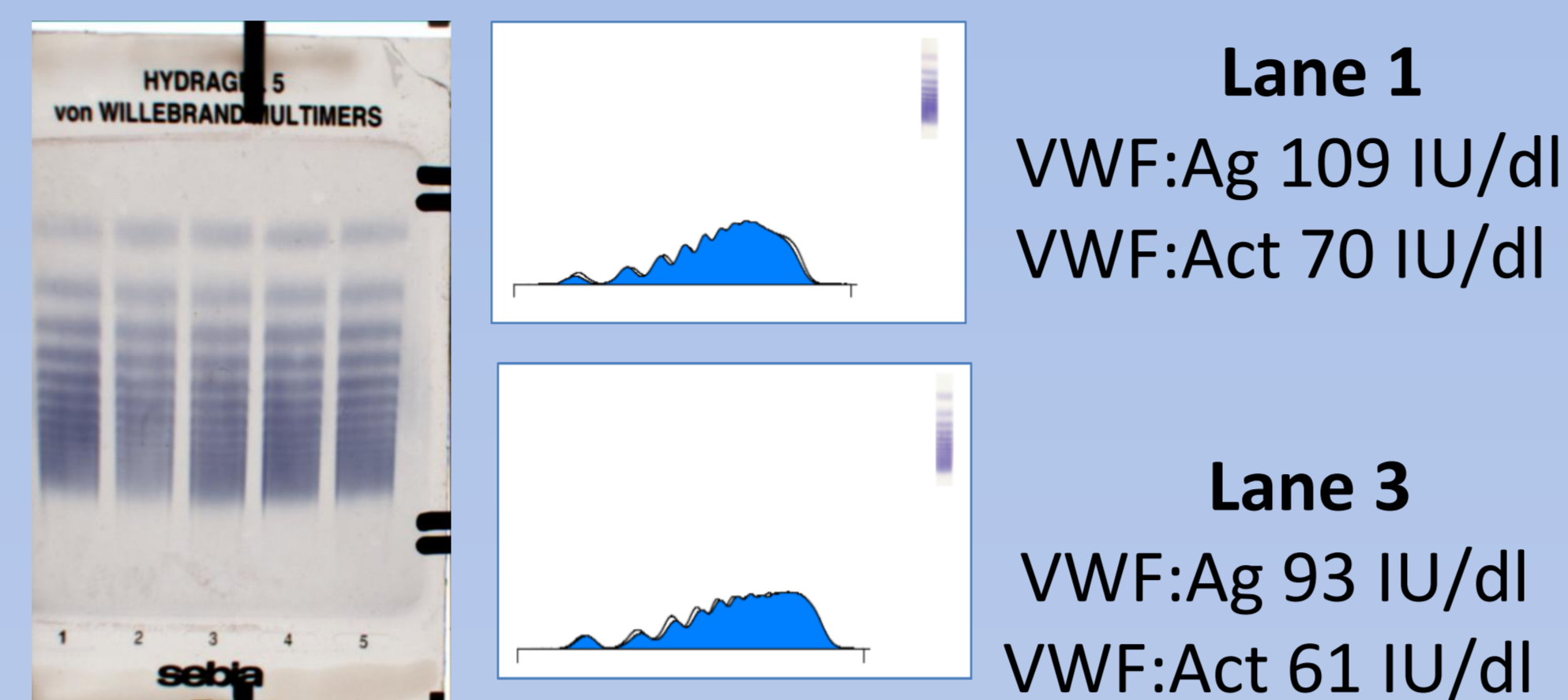


### Type 2M : n= 10

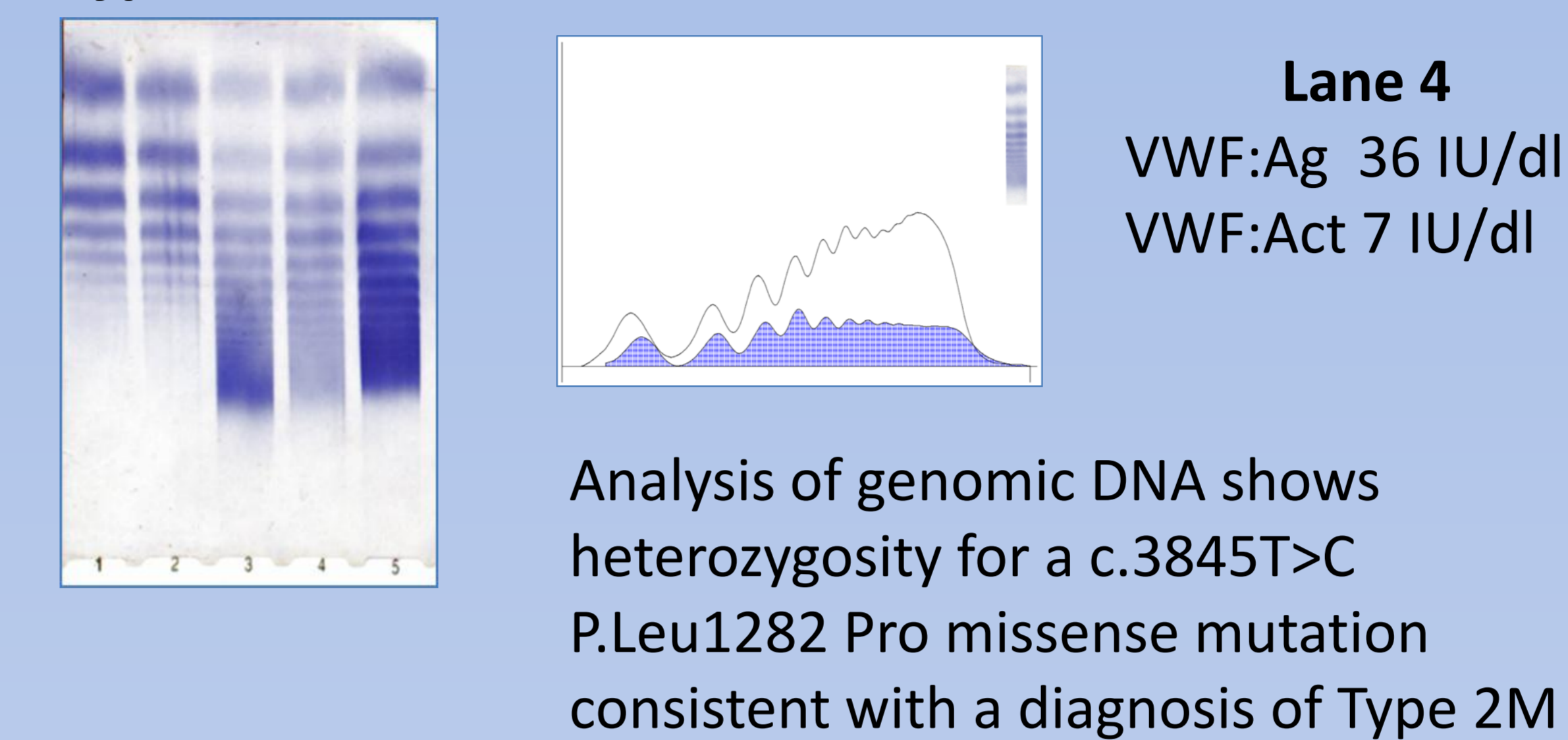


### Type 1 : n=16

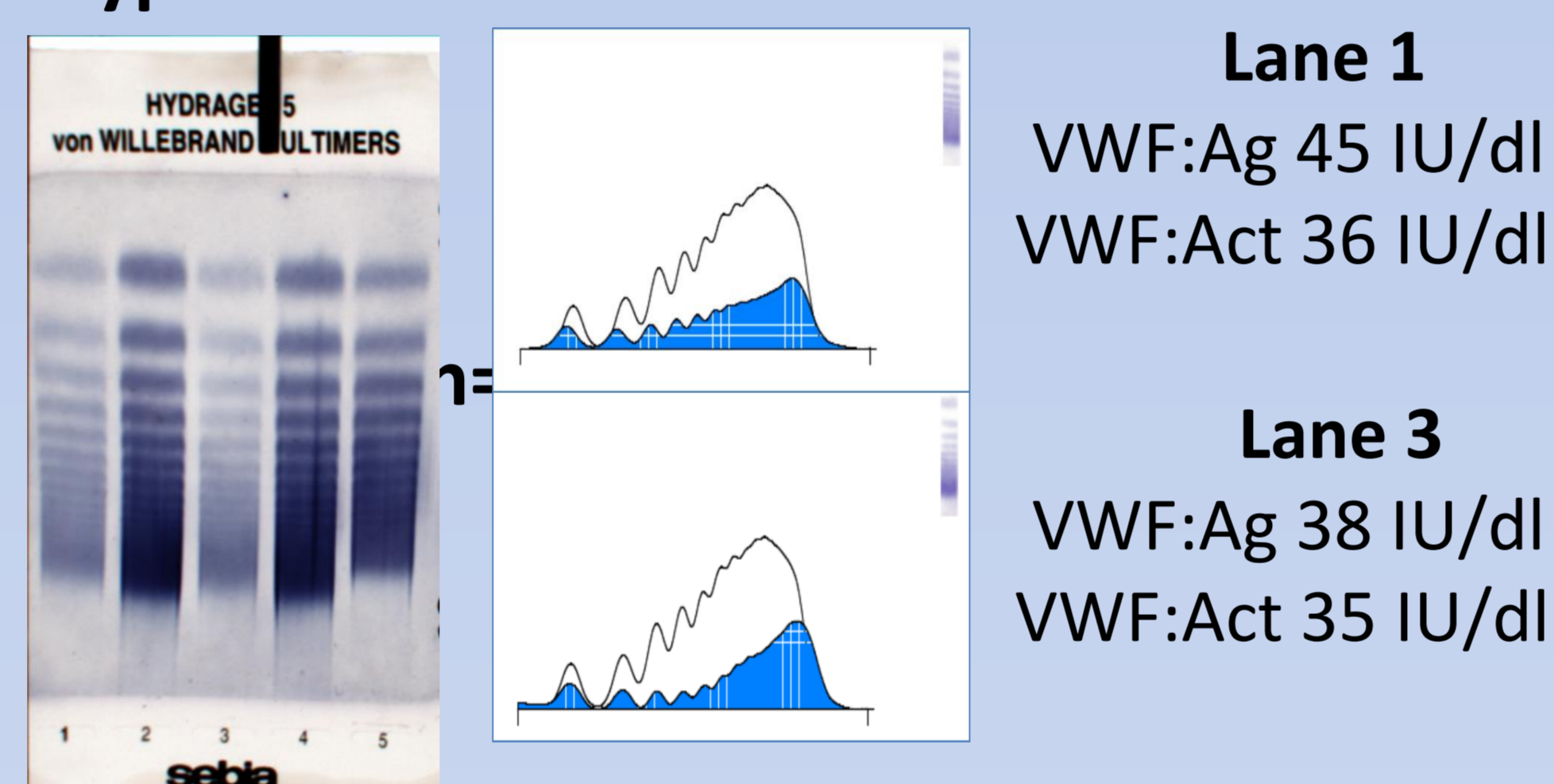


### Thrombotic thrombocytopenic purpura (TTP) : n=4

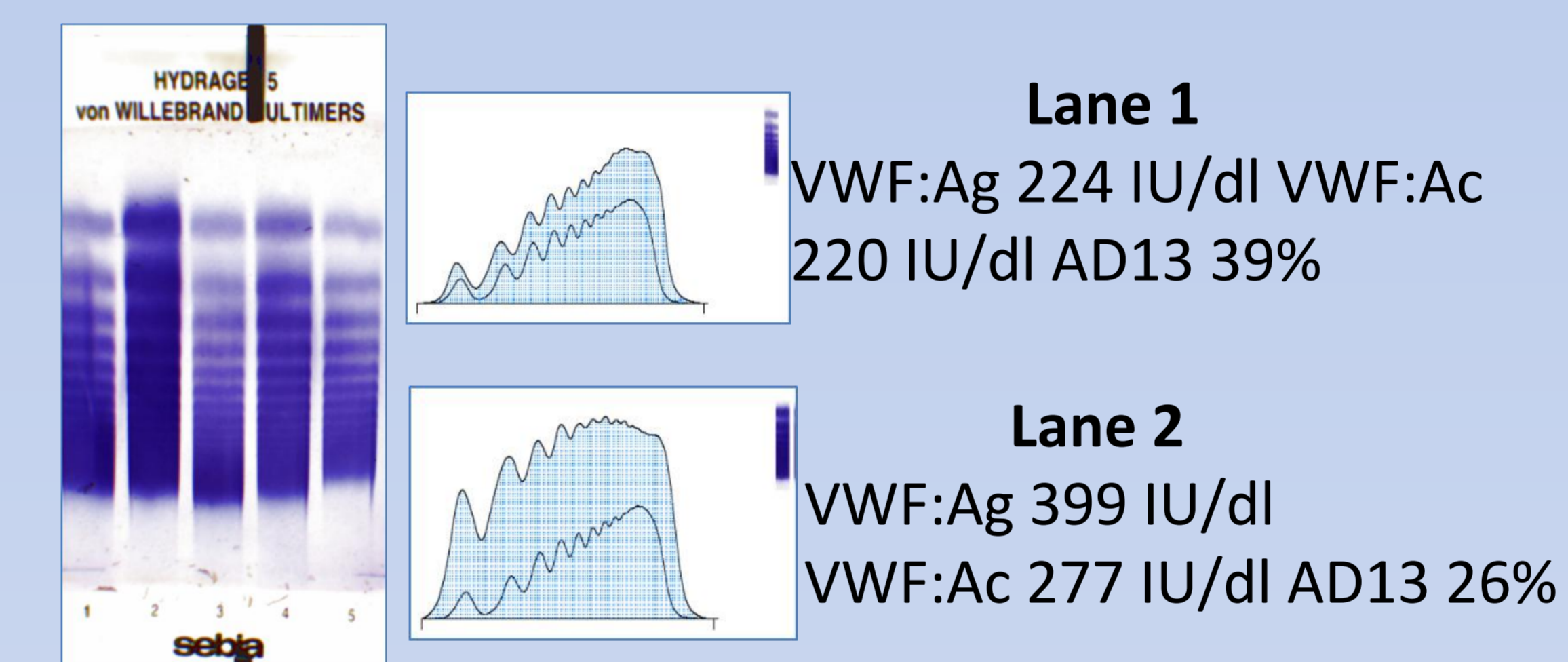


### Type 2A : n=15

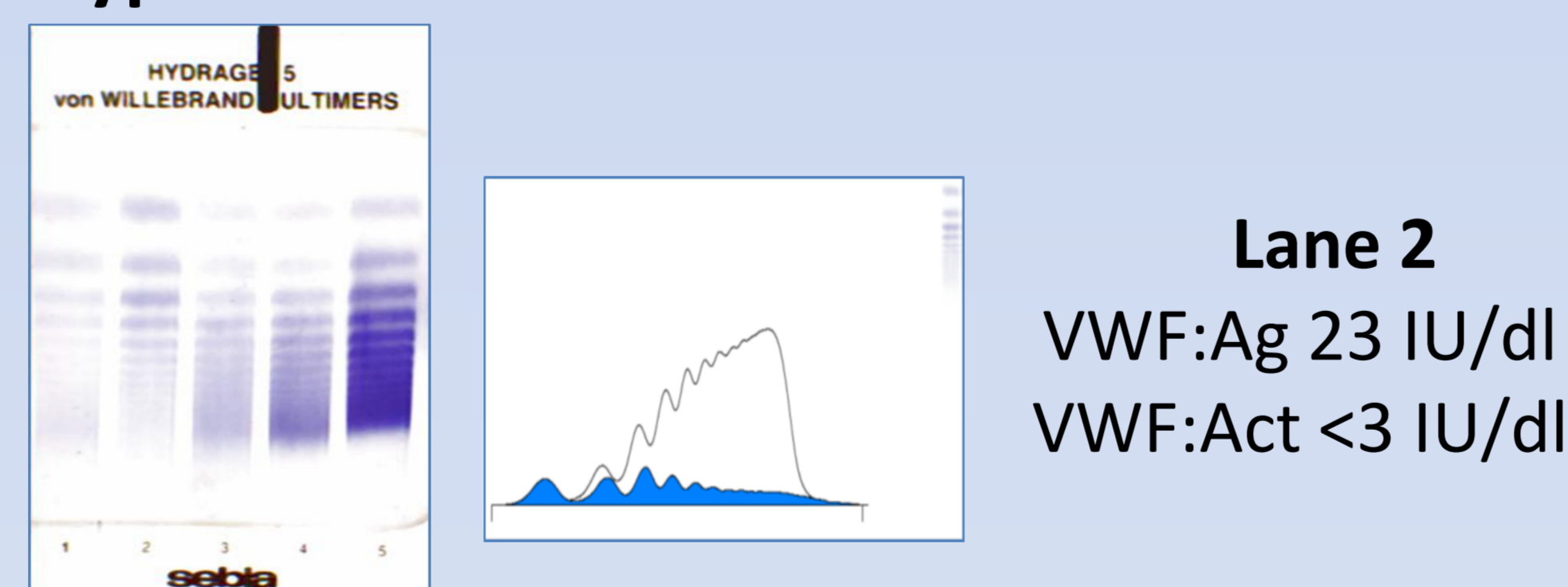

### Type 2B : n=3

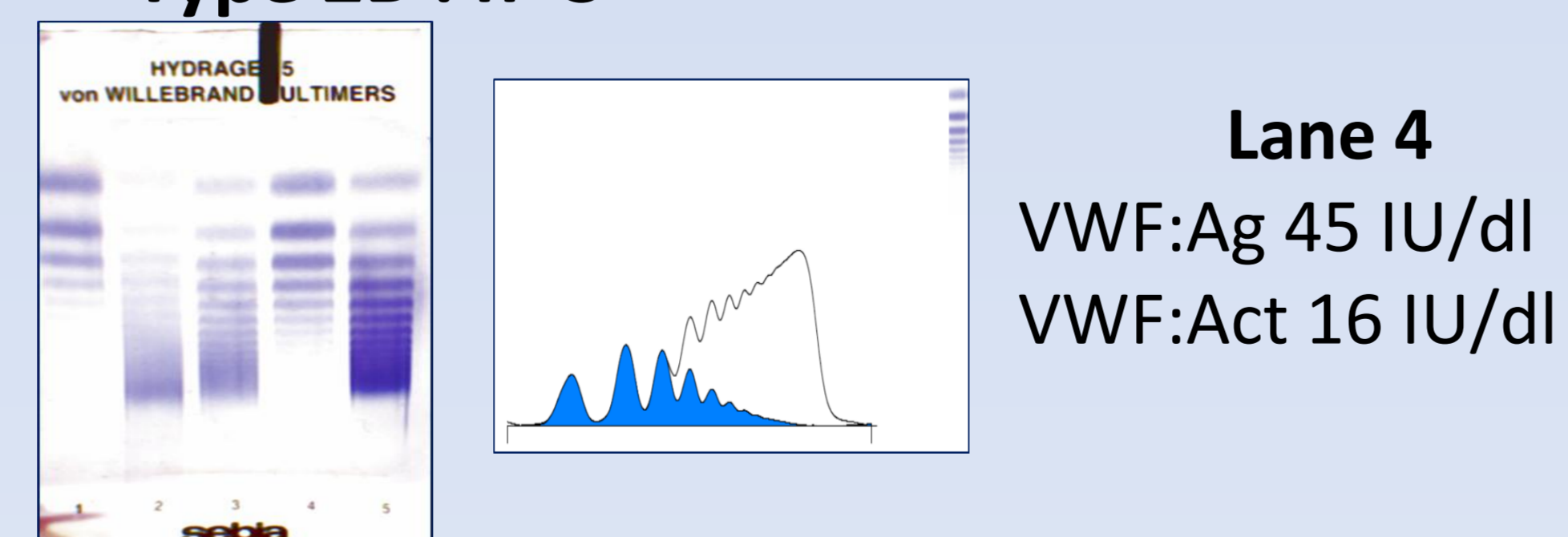

### Type 1 : Pre and Post DDAVP : n=2

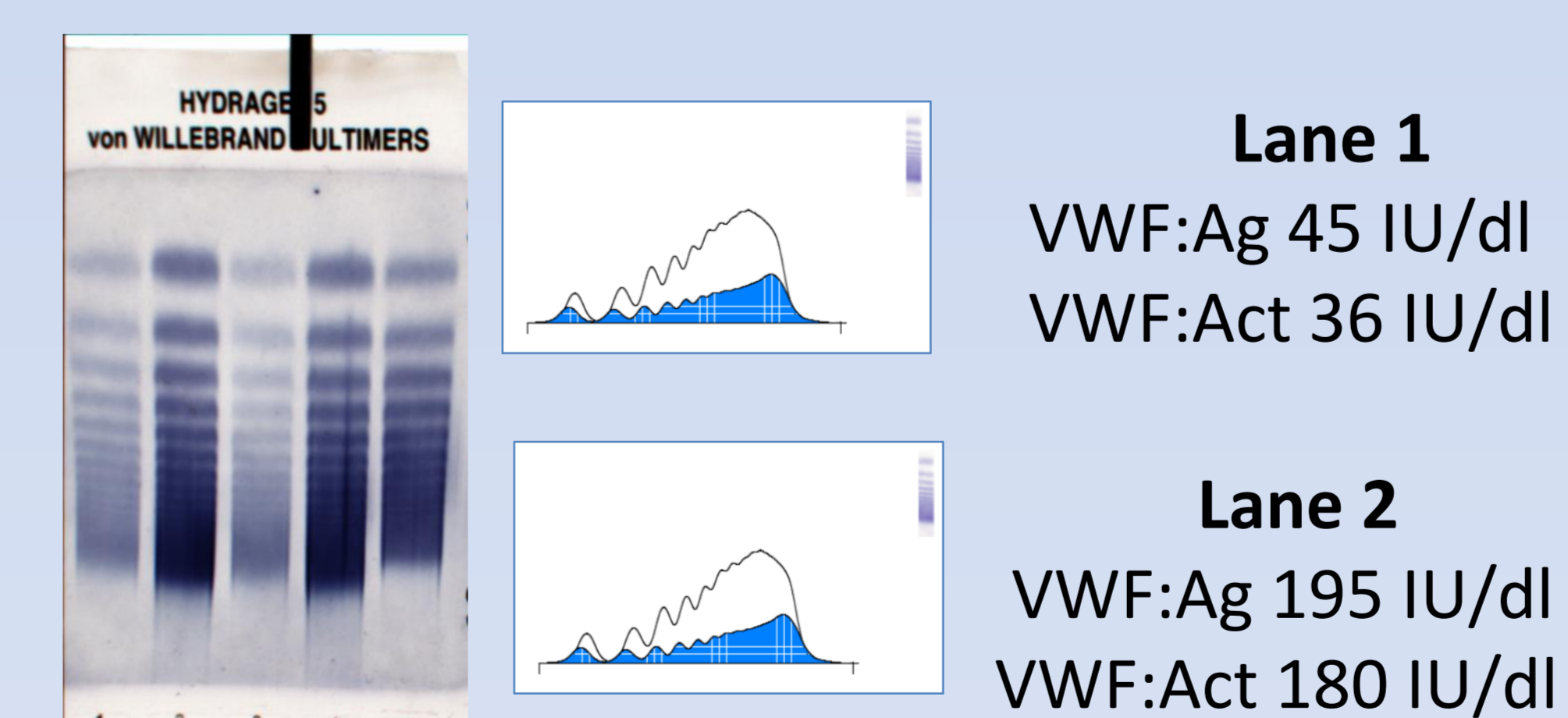

Type 2N : n=2 and Acquired VWS : n=4 also completed (data not shown)

## SUMMARY

The Hydrigel 5 von Willebrand Multimers and Hydrasys 2 Scan instrument :-

- Ready to use gel and reagents are provided
- Results are obtained within the working day
- Clear reproducible bands (no triplets) are obtainable
- Densitometry improves interpretation of gels
- Excellent screening gel

The Hydrigel 5 von Willebrand multimers technique correlates with the in-house technique for :

- Normal samples
- VWD Type 1
- VWD Type 2A
- VWD Type 2B
- Acquired VWS
- TTP

For 6 /10 of the VWD Type 2M a slight decrease of HMWM has been found in the Hydrigel system in contrast to the in-house method where all HMWM appear normal.

## CONCLUSIONS

Our in-house 4 day labour-intensive multimer method is prone to interpretation difficulties due to the lack of densitometry.

The hydrigel system which included all phases of analysis : sample application, migration, incubation, staining/destaining, drying and densitometry on one instrument saved on staff time and allowed for easy interpretation of results.